CE

User manual
Gebrauchsanleitung
Mode d'emploi
Gebruikershandleiding

3.03
03/2015

TRACK 

wheelz ahead[®]

LET'S GET MOVING

User manual	2
Contents	3
1 General information on your rollator	4
2 Technical specifications	7
3 Guide to safe use of your rollator	8
4 How to operate your rollator	11
5 Warranty	17

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Congratulations on your WheelzAhead® rollator purchase!

Before you start using the rollator we ask that you, and your accompaniment, carefully read this manual. If you have questions about the rollator or the content of this manual, please get in touch with WheelzAhead®.

The production process of your rollator has been TÜV certified and adheres to the ISO 13485 regulations.

WheelzAhead® hereby declares that your rollator adheres to the European guidelines for medical support products 93/42/EEG and 2007/47/EEG.

© Copyright 2013 WheelzAhead® BV

The information in this document can be changed or modified at any time and without prior notice. WheelzAhead USA, LLC cannot be held responsible for inaccuracies, typographical errors or photographic errors in this document.

Contents

1	General information on your rollator	4
2	Technical specifications	7
3	Guide to safe use of your rollator	8
4	How to operate your rollator	11
5	Warranty	16

1 General information on your rollator

What have I purchased?

You have purchased a foldable, light weight rollator. This rollator is meant for both indoor as well as outdoor use for those that are mobile to a lesser degree.

Where can I find the rollator's serial number?

On the cross bars underneath the seat you will find a sticker with the serial number of your rollator on it.

What is the maximum allowed weight my rollator can carry?

The maximum allowed weight is 135Kg. This includes any luggage in the designated compartments that are being carried with the rollator.

When do I use the rollator?

The TRACK> rollator supports people who are limited in their mobility indoors, while shopping or walking outdoors.


The main goal of the TRACK> rollator is to make people who are limited in their mobility more mobile.

What does the rollator consist of?

Warning!

To avoid danger of suffocation, keep this plastic bag away from babies and children. Do not use this bag in cribs, beds, carriages or play pens. This bag is not a toy.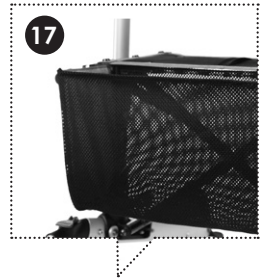

- | | | | |
|----|-------------------------------------|----|-------------------------------|
| 1 | Ergonomic handlebars | 12 | Reflectors |
| 2 | Hand brake / Parking brake | 13 | Front wheels, locking ability |
| 3 | Adjustable and removable back strap | 14 | Rear wheels |
| 4 | Walking stick holder | 15 | Elevation pedal |
| 5 | Lamp | 16 | Wheel brake |
| 6 | Seat | 17 | Mesh bag |
| 7 | Adjustable bagholder | 18 | Shoulder bag |
| 8 | Cross bars | 19 | Travel bag |
| 9 | Height adjustment | 20 | Pouch |
| 10 | Frame lock | 21 | Insert for name tag |
| 11 | Castor wheels | 22 | Trayplate |

2 Technical specifications

TRACK

Length	700 mm / 27.90"
Total width	635 mm / 25"
Width between the handles	500 mm / 19.68"
Height	805 - 935 mm / 31.69 – 36.81"

Measurements when folded	680 x 250 x 810 mm 26.49" x 9.27" x 31.57"
--------------------------	---

Height of seat	560 mm / 22.04"
Width of seat	460 mm / 18.11"
Depth of seat	225 mm / 8.85"

Total weight with accessories	8,5 Kg / 17.64 Lbs
Total weight without accessories	7,4 Kg / 15.43 Lbs

Maximum load bearing capacity	5 Kg / 11 Lbs
-------------------------------	---------------

Materials

Frameparts: Aluminium anodised

Connecting parts: Polyamid (PA6) glasfiber reinforced

Brake handles: Polyamid (PA6) glasfiber reinforced

Back belt: Plastic (Polyester), EPE foam, mesh nylon

Wheels: PU (Tire), PP (Rim)

Seating: Mesh nylon

Handles: Polyamid (PA6) glasfiber reinforced with PVC

Tray: Polycarbonate (PC)

3 Using your rollator safely

Should I check anything before use?

Always make sure that the following questions can be answered affirmatively before using the rollator:

- Are the brakes working properly?
- Is the frame lock properly attached?
- Is the seat fully unfolded?

When can I take part in traffic?

It is important that you practice using the rollator well before taking part in traffic.

Would adjustments to the rollator affect its safety?

Adjustments may negatively influence the safe use of the rollator.

WheelzAhead® will no longer vouch for the rollator's quality if adjustments are made and the warranty will be void.

When is my rollator at risk of tipping over?

This is possible in the following situations:

- You are on a slanting surface
- When riding onto and off sidewalks
- On irregular ground, around drains for instance
- You are over-reaching to grab something in front of, behind or next to the rollator

Practice walking with the rollator on a slanting surface as well as getting on and off sidewalks. By shifting the weight of your upper body slightly forwards or backwards, you change the rollator's center of gravity. This way you can safely mount a small obstacle. Should you be in doubt, never attempt to do this without assistance.

When do I put on the parking brakes?

You should always secure the parking brakes in the following cases:

- When you are sitting down on the rollator
- When you are getting out of the rollator
- In a lift or on a wheelchair lift
- When stopping on an uneven surface

How do I adjust the brakes?

This is essentially the same process as adjusting the brakes of a bicycle. On the bottom of the frame, close to the rear wheel, you will find the end of the braking cable. By turning the screw at the end, you can adjust the braking cable, tighter or looser. Should you have any doubts about the tension of the braking cable or about general brake functioning, please cease using the rollator until you have consulted a professional.

What should I consider when reaching for item in front of, next to or behind the rollator?

The main thing to take into account is never to shift the center of balance too much when reaching for items; this may tip over the rollator. Always pay attention to the following:

- Place the rollator as closely as possible to what it is you want to pick up
- Make sure the castor wheels are aimed forward and secured
- Put the rollator on its parking brakes

How do I get up and down stairs?

Do not attempt to go up or down stairs while leaning on the rollator. Fold up the rollator and carry it up or down. Neither attempt to use the rollator on an escalator.

How to manage or navigate a step or curb?

The foot lever helps you to overcome a step or curb.

First you roll the TRACK> up against the step or curb with both front wheels. Then you push with one of your feet down on the foot lever while gently pulling back on the handle bars. The front wheels will now lift from the ground. Make sure you are in a stable position while doing this. Now push the TRACK> forward, over and on the step or curb. Once the front wheels are on the higher surface, you can lower them and regain support. Keep pushing the TRACK> forward until the rear wheels are also securely on the elevated surface. In case you doubt your ability to overcome such an obstacle, wait for help or assistance. Never try to overcome an obstacle on your own in such a case.

For whom is the TRACK> not suited?

The TRACK> is not meant to be used by people who:

- Have severe balance disorders
- Have severe perception disorders
- Have problems while sitting

Any other tips?

- If you leave the rollator out in the sun for some time, the metal parts may heat up considerably
- Bright and/or reflective clothing will much increase your visibility in traffic
- Be careful with fire and burning cigarettes



4 How to operate your rollator

How do I unfold the rollator?

Your rollator is ready for use in 3 steps:

- 1 Place the rollator endwise and press the 2 red clips inwards one by one
Now you can move the back wheels of the rollator towards the ground
- 2 Secure the frame lock
- 3 Unlock the front wheels
- 4 Separate the handles and push the seat down until it clicks in place.

Note: The frame lock needs to be tightened before the rollator can be unfolded.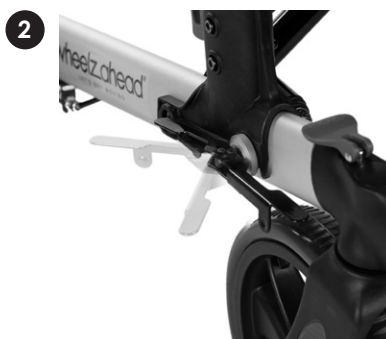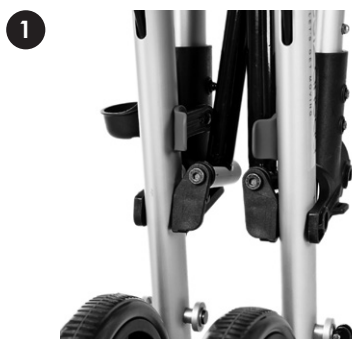


How do I fold the rollator back up?

The rollator is as easy to – quite literally – double up as it is to unfold it:

- 1 Lift the seat by the loop so that both sides of the rollator come together and secure them with the red clip
- 2 Secure the front wheels
- 3 Open the frame lock by taking the handle out and moving it forward
- 4 Now you can move the handle bars toward the rear wheels and you will hear 2 confirming clicks.

Note: The frame lock needs to be loosened before the rollator can be folded.

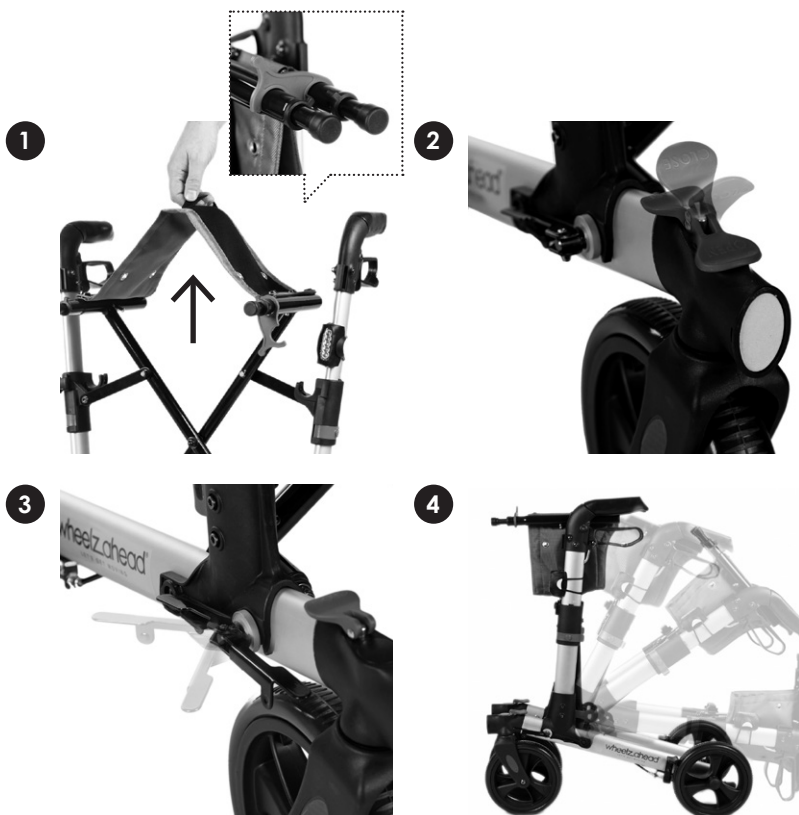How do I adjust the handles in height?

On the outside of your rollator's frame you will find 6 different positioning options for your handles. To come to the correct height please mind the following:

- Stand between the rear wheels
- Try to stand as upright as possible
- The arms should hang loosely from relaxed shoulders
- The handles are to be set at wrist height
- The elbows should be lightly bent when you hold the handles
- Please make sure to always have the handlebars at the same height for improved stability.

Note: Do not adjust the handles when the rollator is folded up twice.

How to secure the mesh bag?

The attachment points for both the shoulder bag and mesh bag are the same. To secure the mesh bag, pull out both attachment points. They will automatically stop at the correct length. Now secure the bag on both sides by putting the loops of the bag around the attachment points on the TRACK>. Pull on the black handles from the mesh bag to secure the velcro to make sure the bag will stay in place. Your mesh bag is now ready to be used.

Note: when you have secured the mesh bag, please be sure to empty it completely before folding the TRACK> rollator.

How do I place the tray plate?

Underneath the tray plate you will find 4 pins. These pins fall into the ringed holes in your seat. By placing the pins into the ringed holes you prevent the tray plate from sliding off your seat while using the rollator. Do always check if the tray plate is placed correctly before using your rollator with tray plate.

What is the proper posture when walking behind the rollator?

In order to come to the right posture behind the rollator, please mind the following:

- Try to walk as upright as possible
- Stay close to the rollator when using it
- Do not lean on the handles too much
- Make sure you can reach and apply the brakes at all times
- Prevent a crooked posture due to fatigue

When tired, sit down for a moment and recuperate. You need enough balance and strength in both arms and legs to be able to walk around with a rollator, you also need a pair of sturdy (laced up) shoes and proper eye sight. Wear glasses if necessary.

What should I pay attention to when walking with the rollator inside or outside?

Walking with the rollator inside requires the following:

- Make sure you have enough space to move around; have furniture moved if necessary
- Remove any unsecured rugs and thresholds
- Sufficiently light all rooms that you enter (including the hallway)

Walking with the rollator outside requires the following:

- Beware of loose tiles and stones on sidewalks
- Properly lift your legs to prevent tripping
- Opt for having your fingers on the brakes at all times so you can brake immediately should you trip
- Make as much use as possible of wheelchair provisions on the edges of sidewalks when getting on or off pavements
- Take a rest when necessary (make sure to secure the brakes first)

Also take into account that you may need a bit more time crossing the road.

How do I use the parking brakes?

By moving the brake handle down until it clicks into place, you secure the rollator's parking brakes. The rear wheels of the rollator are now secured. When you move these handles back up, you deactivate the parking brakes.

How do I lock and unlock the front wheels?

On top of the frame above each wheel fork you will find a red clip which you can move forward 'OPEN' and backward 'CLOSE'. By placing the clip in the 'OPEN' position you unlock the front wheels and the front wheels will move along with the rollator. By placing the clip in the 'CLOSE' position you will lock the front wheels and the front wheels will not move along with the rollator.

How am I to take care of my rollator?

Weekly:

- Check if the brakes are still functioning properly

Every 3 months:

- Check if the cross frame still allows for easy unfolding
- Check if all bolts and screws are secure
- Loose screws have to be fixed and secured with Loctite 435.
Please adhere the produces user manual
- Check the tire surfaces for wear, tear and any uneven areas
- Check the front and rear wheels for wheel clearance

All hinged parts are easily kept functioning smoothly by lubricating them regularly.

How do I keep my rollator clean?

With a mild soap and water you can keep textiles, frame and polymer parts clean. We also recommend you to wax the frame from time to time.

Never use:

- Abrasive products
- Pressure cleaner
- Steam
- Abrasive or corrosive wax
- Aggressive chemicals
- Silicone spray

Always dry the rollator carefully in case it got dowsed in water or rain.

Re-use

The TRACK> rollator can be re-use by others. Before re-use the rollator must be thoroughly cleaned en disinfected. The rollator must be checked for user safety and approved by an authorised dealer.

5 Warranty

1. WheelzAhead® warrants that, during the Warranty Period, the Product will, with normal use and service, be free from faulty parts, manufacture defects or workmanship errors.
2. The TRACK> Rollator comes with a 24 month warranty on the frame and the cross bars, starting from the date of purchase by the original buyer. Any manufacturer defects (production errors or material failure) also carry a warranty of 24 months.
3. This warranty is valid only for Products that are purchased new and unused directly from WheelzAhead® or from authorized third party resellers or distributors.
4. The Customer must provide the original proof of purchase and the serial number of the Product which demonstrates compliance with the above conditions to receive any services under this warranty.
5. The warranty is non-transferable.
6. This warranty only applies where a defect has arisen, wholly or substantially, as a result of faulty manufacture, parts or workmanship during the Warranty Period.
7. The warranty does not apply where damage is caused by other factors, including without limitation:
 - A normal wear and tear;
 - B abuse, mishandling, accident or failure to follow operating instructions;
 - C servicing or modification of the Product other than by WheelzAhead®, or their authorized service agents;
 - D use of the Product with other accessories, attachments, product supplies, parts or devices that do not conform to WheelzAhead® specifications; or
 - E shipment or other transit.
8. This warranty does not cover:
 - A the replacement of any consumables such as tires, brake pads, brake cables, textiles, etc.
 - B repair or replacement of any Product damaged during transit to or from WheelzAhead® ; or
 - C any loss or consequential damage incurred while the Product is being repaired.
9. Instructions for making a warranty claim are included in the enclosed materials. If the warranty claim is not accepted:
 - A WheelzAhead® will inform the Reseller, Distributor or the Customer in case of a purchase directly from WheelzAhead®;

- B if requested to do so by the Customer, the Reseller, Service Agent, Distributor or WheelzAhead in case of a purchase directly from WheelzAhead®, will repair the Product provided you pay the standard charges for such repair; and
- C if applicable the Customer will be responsible for all costs associated with collection of the Product from the Reseller, Service Agent, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead®.

Making a warranty claim

Please contact your Reseller, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead®, by phone or email to make a warranty claim.

- A You must inform your Reseller, Distributor or WheelzAhead in case of a purchase directly from WheelzAhead® as soon as the warranty claim arises.
- B You must send to your Reseller, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead®, your original proof of purchase, your serial number and your written description of the fault (including image samples and any other relevant material); and
- C Subject to the applicable consumer laws in your jurisdiction, you must pay for all packing, freight and insurance costs for transit of the Product to your Reseller, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead®.
- D If the warranty claim is accepted your Reseller, Distributor or WheelzAhead® in case of a purchase directly from WheelzAhead® will, subject to the applicable consumer laws in your jurisdiction, at its cost: repair or replace any faulty parts or rectify any faulty workmanship; and return the Product to you.

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© Copyright 2013 WheelAhead® BV