

# Chamber Brite™ - Powdered Autoclave Cleaner

**Tuttnauer**™  
Your Sterilization & Infection Control Partners

# Chamber Brite



## No Mess, No Hassles, No Measuring...

Chamber Brite powder is simply the easiest way to keep autoclaves clean and free of water deposit build-up. Designed specifically for steam sterilizers, Chamber Brite effectively cleans and descales the reservoir, chamber and tubing in minutes.

- Convenient, pre-measured packets
- Cleans reservoir, chamber and tubing
- Uses less storage space
- Replaces messy liquid cleaners



# Chamber Brite™ - Powdered Autoclave Cleaner

## Few simple steps to a hassle-free clean



Cleaning Instructions:\*

1. Sprinkle Chamber Brite powder along the bottom of the COLD autoclave chamber.



2. Start a sterilization cycle WITH WATER and NO DRYING CYCLE



3. Drain solution from reservoir and discard.

4. Fill reservoir and repeat steps 2 and 3 without Chamber Brite.



5. Wipe inside of chamber with damp cloth or sponge.



6. Fill reservoir with distilled water and sterilizer is ready for use.



For best results, clean autoclave every 20 cycles or once per week as needed. It is essential to follow the equipment manufacturer's directions for cleaning and maintenance.

\*For full instructions on how to use Chamber-Brite -follow the directions on the box.



Fully Automatic  
E Series



Manual  
M Series



2540EL

### Tuttnauer Tabletop Autoclaves

#### Tuttnauer Europe b.v.

Hoeksteen 11, 4815 PR, P.O.B. 7191, 4800 GD Breda, The Netherlands  
Tel: +31 (0)76 542 3510, Fax: +31 (0)76 542 3540  
E-mail: [info@tuttnauer.nl](mailto:info@tuttnauer.nl), [www.tuttnauer.com](http://www.tuttnauer.com)

  
Your Sterilization & Infection Control Partners



LTR330-0015/CBF/EUR/08.07