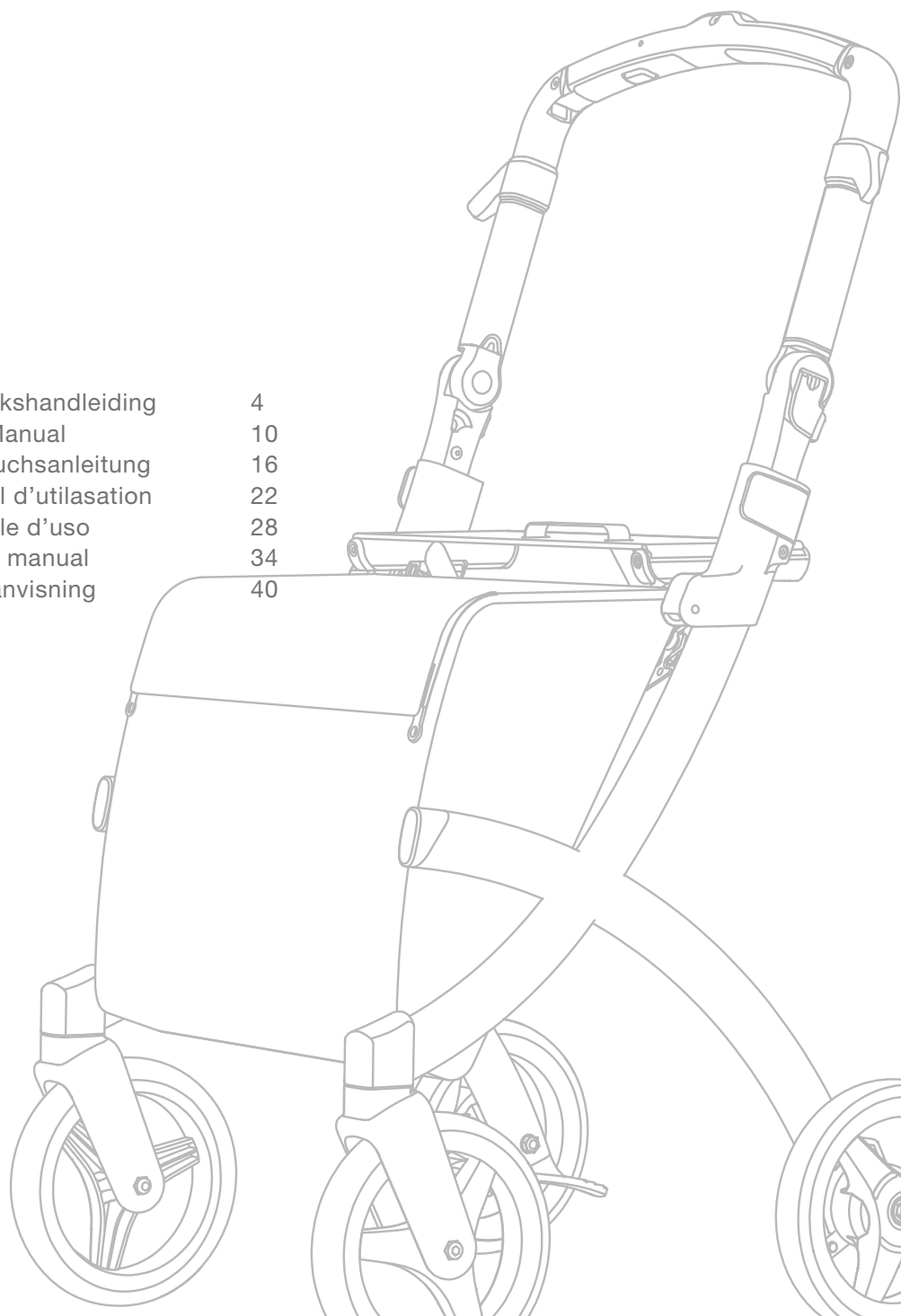
rollz[®]
flex

Handleiding
User Manual
Gebrauchsanleitung
Manuel d'utilisation
Manuale d'uso
Bruger manual
Bruksanvisning

Rolz Flex

NL - Nederlands	- Gebruikshandleiding	4
EN - English	- User Manual	10
DE - Deutsch	- Gebrauchsanleitung	16
FR - Français	- Manuel d'utilisation	22
IT - Italiano	- Manuale d'uso	28
DK - Dansk	- Bruger manual	34
SE - Sverige	- Bruksanvisning	40

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Rollz International BV – Rollz Flex

© Copyright 2016 Rollz ® International BV

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Rollz International BV – Rollz Flex

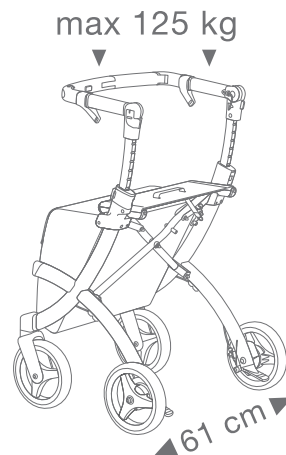
© Copyright 2016 Rollz ® International BV

Information contained in this document may be changed without prior notice. Rollz ® International BV cannot be held responsible for technical faults or defects.

The purchased product may differ from the product described in this user guide. For the most up-to-date user guide, contact Rollz International BV or go to www.rollz.com.

CE-conformity

The product fulfils the requirements of the EU-Directive 93/42 EEC for medical devices. Based on the classification rules in Attachment IX of this directive the product is classified into class I. The declaration of conformity has been done under the own responsibility of Rollz International BV based on the Attachment VII of this directive.

1.1.

Contents

1. Before you begin	12
1.1 Important information	12
1.2 Safety	12
1.3 Warning	13
1.4 Indication	13
1.5 Contra-indication	13
1.6 Check if your delivery is complete	13
1.7 Reuse	14
1.8 Use	14
1.9 Technical specifications	14
1.10 Contact	14
2. Warranty	15
3. Use	46
3.1 Unfold	46
3.2 Fold	46
3.3 Posture	47
3.4 Height adjustment	47
3.5 Push bar adjustment	48
3.6 Operating the brakes	49
3.7 Resting on the Rollz Flex	49
3.8 Thresholds and kerbstones	49
3.9 Bag usage	50
3.10 Bag removal	50
4. Maintenance	52
4.1 Brake adjustment	52
4.2 Cleaning/disinfection	52

1. Before you begin

Congratulations on the purchase of your Rollz Flex. Rollz has written this manual so that you can use the Rollz Flex in the safest and most convenient way. Please have your purchase registered on www.rollz.com or by using the included Registration form. We hope you will enjoy the use of your Rollz Flex.

1.1 Important information

- Read these directions carefully before using the product and keep them safe for future reference.
- Do not compromise your safety and that of others around you by the incorrect use of this product.
- This manual was written for the Rollz Flex.
- You will find the serial and article numbers on the product sticker of your Rollz Flex. Which is located on the backside of the right hinge tube.
- This product has been TÜV certified.
- Only use original or Rollz International BV approved parts in the event of replacement or repairs.
- Use original Rollz accessories only.
- The warranty will be invalid after the product has been damaged through overloading, incorrectly folding in or out, the use of parts which are not approved by Rollz, or the use of accessories other than Rollz accessories. For the full warranty conditions see page 15 of this user manual.
- To keep the Rollz Flex in optimal condition, frequent cleaning is necessary. See chapter 4.2 Cleaning/disinfection on page 34.
- The Rollz Flex has been patented and the brand & model are protected.
- The Rollz Flex is made for people weighing up to 125 kg who are 155-195 cm tall.
- The Rollz Flex bag is made for loads up to 20 kg.
- Store the Rollz Flex in a dry place inside the house.
- Colours of the Rollz Flex parts may fade slightly when exposed to sunlight.
- Do not expose the product to extreme temperatures.

1.2 Safety

- The Rollz Flex is to be used for supporting one person only.
- Always use the push bar in the right position. Read the chapters on height adjustment and posture (page 29) for the correct positions.
- Adjust the brakes in a timely manner to prevent the loss of braking power.
- Only use the Rollz Flex as a rollator when the push bar is fully positioned to the front and in the rollator position. You can find the correct adjustment in chapter 3.5 Push bar adjustment (page 30).
- Position your hands in the middle of the handgrips at all times. You can recognize the correct location by the grooves on the handgrips. See chapter 3.3 Posture on page 29.

1.3 Warning

- The Rollz Flex has been developed for the convenient control of the various operations. However, it is essential to make sure that the user has the physical ability to independently perform all the operations. If not, please call in the assistance of somebody else. Rollz International suggests you check this on a regular basis.
- Keep the Rollz Flex away from open flames and other heat sources, such as electrical heaters or gas heaters.
- Do not use the Rollz Flex if any part has been broken or cracked, or if any part is missing.
- Make sure that all clasps are locked before use.

1.4 Indication

The Rollz Flex is a mobility aid that can be used both as a rollator (only in rollator position) and a shopping trolley. The Rollz Flex in rollator position provides support while walking for people with limited mobility, both in- and outside the house. The seat allows you to take a rest. In case your support need is currently low, you can tilt the push bar to suit your current needs. See 3.5 Push bar adjustment on page 30.

1.5 Contra-indication

The Rollz Flex is not suitable for people with:

- visual perception disorders
- severe balance disorders (Causing the user to be unable to independently operate the Rollz Flex)
- seating problems

1.6 Check if your delivery is complete

The Rollz Flex comes packed and ready for use. Check that all parts have been supplied. The Rollz Flex comes complete with:

- 1 inner bag
- 1 user manual

WARNING!

Keep the plastic packaging materials away from children. Children may play with it and suffocate. Discard the packaging materials without delay.

1.7 Reuse

- The Rollz Flex is suitable for reuse.
- For safe handling, pass all technical documentation on to the new user.
- Before passing the Rollz Flex on to a new user, it should be cleaned, disinfected and checked by the dealer for damage.

1.8 Use

The Rollz Flex comes complete and ready for use. However, the Rollz Flex should be adjusted to meet your personal preferences before use. Carefully read all the points in this chapter before you put the Rollz Flex into operation.

1.9 Technical specifications

- Maximum weight of the user: 125 kg
- Minimum and maximum physical height of the user: 1.55 m tot 1.95 m
- Maximum load bag: 20 kg
- Net weight of the product: 6.9 kg
- Dimensions:
 - Extended -> HxLxW: 80-100 x 58 x 63.5 cm
 - Retracted -> HxLxW: 80-100 x 58 x 32 cm
 - Seat width: 435 mm
 - Seat depth: 300 mm
 - Seat height: 620 mm

1.10 Contact

Rollz International is always keen to further improve its products, we are interested in your opinion. This could be about the use of the product, the manuals or any other aspects. Mail to: info@rollz.com. Or contact Rollz International for any other questions or comments you may have:

Treubweg 51 bg
1112 BA Diemen
The Netherlands
+31 (0)20 362 20 10
info@rollz.com
www.rollz.com

2. Warranty

Rollz International BV, with its registered office in Diemen (hereafter called „Rollz“), in its capacity of manufacturer, provides a warranty for its product called „Rollz Flex“ (hereafter called „the product“), on the following conditions:

1. The warranty period is 24 months from the purchase date. There will be no extension of the warranty period, not even in the case of repairs. In addition, Rollz International offers the first owner a 5 year warranty on materials and structural defects of the Rollz Flex frame, under normal use and maintenance. The general warranty period of 24 months applies to the other parts, such as handles, brake cables, hinge, adjustment handles, wheels etc., with the exception of normal wear and tear.
2. The warranty gives the buyer the right of repairs; explicitly not the exchange or return of a product.
3. At its own expense and risk, Rollz will take care of transport from and to the shop where you purchased the product (hereafter called „the supplier“), but only in the event of repairs covered under warranty.
4. To make a claim under warranty, the faulty product must be returned to Rollz together with: a copy of the proof of purchase, showing the purchase date and a clear description of the complaints.
5. The warranty is not transferable, therefore only the first owner can lodge a claim.
6. The warranty does not prejudice other rights you may have by law, e.g. the right to hold the supplier responsible for any defects.

Warranty claims are not accepted in the following situations:

- After you carried out any changes or repairs yourself (or after you had changes or repairs carried out by others), without the prior written consent from Rollz or the supplier.
- If the defect is the result of negligence, or an accident, and/or a maintenance procedure other than advised in the manual.
- Damage or the removal of the serial number.
- Damage to the product as a result of normal wear and tear.

Important

The warranty period for the product is 24 months from the purchase date. Please contact your supplier in the event of a defective product. Please register your product online by filling out the form on www.rollz.com. Make sure to enter the serial number of the Rollz Flex. You will find the serial and article numbers on the product sticker of your Rollz Flex. Which is located on the backside of the right hinge tube.

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