





1
Fill the Novo Cup with liquid as required.

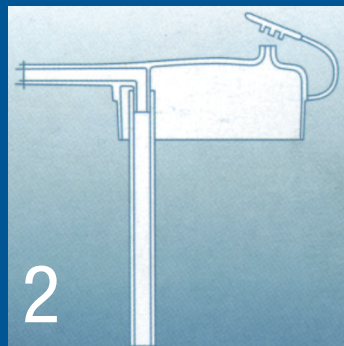

2
Fit the suction tube securely onto the nozzle on the underside of the lid .

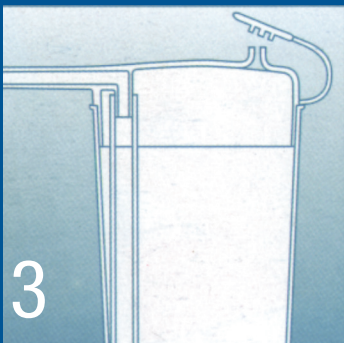

3
Place the lid on the cup and press firmly down on the lid. Do not put the stopper in the air vent or cover it with your hand . If the drink is to be served to a patient in the prone position, place the stopper in the air vent.



4
After the first sip, the cup is spill-proof.

NOVO CUP

Perfect for the bedridden who wants to drink.

The cup allows you to drink comfortably while lying down. You are in complete control of the liquid flow and don't need to be propped up by pillows or helped by carers. Ideal for patients with restricted movements of head and neck.

- Spill-proof when tipped over, which means fewer changes of bed linen
- Takes both hot and cold liquids
- Transparent so contents are visible
- With measurement markings; contains 250 ml liquid
- Dishwasher, microwave and autoclave-safe
- Saves time for the carer who no longer has to lift or support the patient's head. Efficiency in the workplace is therefore optimised and the risk of back or muscle strain minimised

rikacare.dk



DS/EN ISO 9001 CERTIFIED

RIKA PLAST AS, Saunte Bygade 12, 3100 Hornbaek, Denmark
Phone + 45 4975 0130 fax + 45 4975 0201, rika@rika.dk, rikacare.dk