

Somnofit®-S

Anti-snoring mouthpiece thermo-adaptable, reduced volume and optimized comfort

- New patented design.
- Reduce volume.
- Optimized comfort.
- Easier to clean.

SOMNOFIT-S includes two oral mouldings (upper and lower) which can be heated and moulded to fit your teeth. A strap connects the mouldings to hold the lower jaw forward. The user then has a product that is fully personalised and can be worn every night.



8 straps of different lengths are supplied for an adaptation perfectly fitted to the needs of the patient.

Contents

Ref. S-03

- An upper moulding with 1 front hook.
- A lower moulding with 2 back hooks.
- A toothprint plate.
- A floating thermometer.
- A ventilated storage box.
- A set of supple straps.
- Instructions.


Patented medical device

CE



Principle of operation

During sleep, SOMNOFIT-S hold the lower jaw forward by a few millimetres. This advancement clears space around the back of the tongue to allow inhaled air to flow freely.

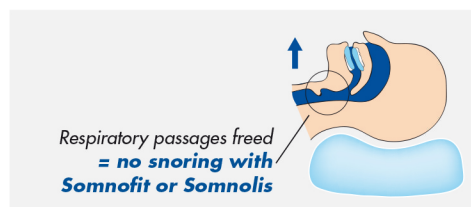

Indications

- Snoring.
- Mild to moderate obstructive sleep apnoea (medical supervision required).
- Obstructive sleep apnoea at certain times: travel, camping, on a boat, holidays (medical supervision required).

Contraindications

- Wear a removable or partial dental prosthesis.
- Have mandibular joint pain.
- Have loose teeth or advanced periodontal disease.
- Upper or lower jaw with less than 10 teeth.
- Are under 18 years old.

Effectiveness

The principle of mandibular advancement is clinically proven. It allows reducing or eliminating snoring in more than 80% of the cases. In the case of obstructive sleep apnoea, the average reduction of the apnoea index is approximately 55%, but can significantly vary from one patient to another. Consequently, an effectiveness examination should be conducted with the medical specialist.